CoaguChek XS PT Controls

REF | 4
04696522 | CoaguChek XS Plus/ XS Pro

SYSTEM
CoaguChek XS Plus/ XS Pro

English

Intended use

CoaguChek XS PT Controls are used for system checks and quality control testing of prothrombin time^{a)} with the CoaguChek XS Plus or CoaguChek XS Pro meter and CoaguChek XS PT test strips.

The CoaguChek XS PT Controls are intended for use for professional users.

a) Also known as Quick's test or Quick's value or under the abbreviation PT/INR

Reagents - working solutions

Each bottle of control contains lyophilized anticoagulated rabbit plasma. Each diluent-filled dropper contains calcium chloride and preservatives in water.

Target values and ranges

The attached code chip contains all relevant lot-specific information to perform a quality control test.

The CoaguChek XS Plus or XS Pro meter displays the control range, the reading obtained and an evaluation of the result (inside or outside of the control range). The result is automatically saved as a control reading in the memory of the meter.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

Follow the applicable government regulations and local guidelines for quality control.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines.

Safety data sheet available for professional user on request.

Handling

Dissolve carefully the contents of one bottle by adding the contents of 1 dropper and allow to stand closed for at least 1 minute to reconstitute. Mix carefully by gently swirling the bottle.

Storage and stability

Store at 2-8 °C.

The lyophilized control plasma is stable up to the stated expiration date.

The reconstituted control material is stable for 30 minutes.

Materials provided

- 4 bottles of lyophilized control plasma Level 1
- 4 diluent-filled droppers
- 1 code chip

Materials required (but not provided)

- REF 04800842, CoaguChek XS Plus meter or
- REF 05530199, CoaguChek XS Pro meter
- REF 04625358, CoaguChek XS PT test strips

Assay

1. Prepare the meter and a test strip according to the instructions given in the operator's manual and in the instructions for use for the test strips.
2. Insert the quality control code chip in the meter.
3. Use the dropper to apply one hanging drop of the control material to the sample application area of the test strip. Save the remaining control solution until the test result is obtained, and for repeating the quality control check if necessary. Use the control material within 30 minutes of reconstitution.

System checks

The CoaguChek XS Plus or CoaguChek XS Pro meter has a number of inbuilt system checks. Please refer to the operator's manual for details. The CoaguChek XS PT test strip has an inbuilt quality control function.

Reproducibility

Precision: 2.8 % CV for an INR of 1.9

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

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COAGUCHEK is a trademark of Roche.